



WC9000+ Pre-Test (PBT – Personal Breath Test) Feature


To access the PBT function, tap the oval button shown below.

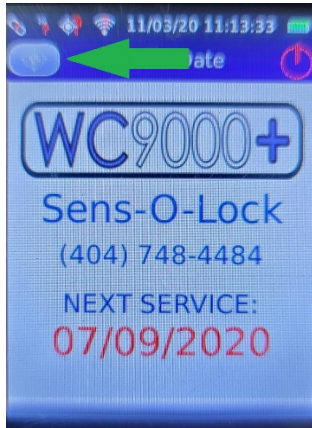

If the feature is not enabled the second option will be blank, as shown below

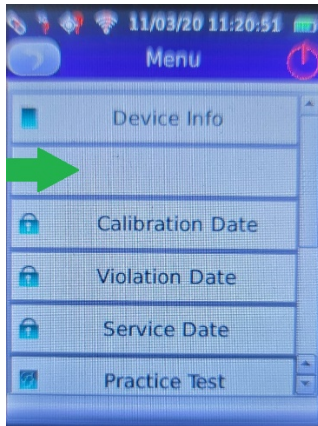

If not enabled, contact Sens-O-Lock of America at 404.748.4484, press 3 for Accounting as ask that the feature be added to your account. Then return to Service Center so the device firmware can be updated and feature added.

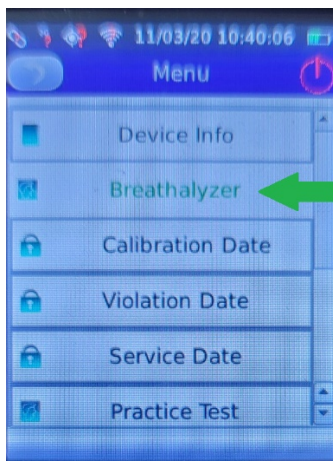

Once the feature is added, choose the Breathalyzer option from the second menu, as shown above, to provide a pre-test. The Pre-Test result will be displayed and stored on the unit but not reported to DMV/court. Be aware that a successful Pre-Test, even if the result is lower than the state fail point, will not allow you to start your vehicle. You will need to return to the main menu and proceed with a normal breath test to start your vehicle.

Note - as a precaution, if the Pre-test level is within .003 of the pass/fail point in your state, we advise waiting, take several deep cleansing breaths and/or rinse thoroughly with water to be sure there's no contaminant present that could cause a fluctuation and elevate your breath level to reading greater than the pass/fail point for your state.